---

## maruvest *Microfein*

---

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**For further Informations contact us !**

# maruvest *Microfein*

Precision Carbon-free Phosphate  
bound Investment

## maruvest-*Microfein*

Is a high heat investment suitable for all Dental alloys. Due to the very fine grain structure, **maruvest-*Microfein*** gives clean, bright castings, which are precise and easy to deinvest.

**maruvest-*Microfein*** offers high thermal stability and problem-free castings.

**Mixing ratios:**

12-14 ml liquid : 60 g **maruvest-*Microfein*** powder (sachet)  
36-38 ml liquid : 160 g **maruvest-*Microfein*** powder (sachet)  
22-24 ml liquid : 100 g **maruvest-*Microfein*** powder

Add powder to liquid and mix under vacuum for 75-90 seconds Allow to bench set for 30 minutes, Place in a cold furnace, burnout at 270-280°C and hold for 20-30 minutes (according to ring size), Hold the final end temperature for 30-60 minutes (according to ring size)

**Never put the hot muffel in cold water, cool down slowly at room temperature otherwise the oxyd-layer can stick to the metal.**

**Casting temperatures**

Precious and Semi-Precious alloys: 600°C - 880°C  
Non-Precious: 860°C - 950°C  
Please also observe the instructions of the alloy manufacturer

**Storage:** **maruvest-*Microfein*** to be stored in a dry and frost-free room for up to 8 months,  
**Caution:** **Investments contain quartz-wear adequate protection-do not breath in dust**

## Expansion properties

| Liquid concentration | Setting-Expansion | Thermal-Expansion | Total-Expansion  |
|----------------------|-------------------|-------------------|------------------|
| <b>100 %</b>         | <b>1,4-1,6 %</b>  | <b>1,2 %</b>      | <b>2,6-2,8 %</b> |
| <b>50 %</b>          | <b>0,4-0,5 %</b>  | <b>1,1 %</b>      | <b>1,5-1,6 %</b> |

## Suggested maruvest liquid concentrations

|                                                         |              |
|---------------------------------------------------------|--------------|
| Precious alloys<br>Inlays, Crowns, Bridges etc          | 65 % - 85 %  |
| Non-Precious alloys<br>Crowns, Bridges, Telescopic work | 95 % - 100 % |

All figures are approximate

The above are given as a guide only and may be varied according to individual technique. When diluting **maruvest-Liquid**, than use only distilled water.

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