megasol NF

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megasol NF

Separating agent, based on alginate
For isolating plaster against resin

megasol NF is used to isolate denture resin against plaster. It holds on to the plaster perfectly, thus preventing the formation of skin and layers.

Processing:

Dried out models and precasts have to be watered in warm water until the air is completely removed from the plaster and replaced by water.

Allow the surface of the plaster to dry.

Apply a thin coat of separating agent twice to the watered plaster parts using a soft brush. Remove excessive material from pores and cracks. Artificial teeth and clasps should not get in touch with the separating agent.

Allow the insulating coat to dry completely. Well dried parts appear as a shiny surface. Non shining surfaces need to be insulated again.

Recommendations: The surfaces that need to be insulated should be free of wax, grease or separating agent. Otherwise the insulating coat may not stick or it may crack. In those cases it is recommended to thoroughly boil out the plaster parts.

After opening the container, it must be stored in the refrigerator!

Never apply the insulating material with the brush directly from the container. Always fill a small portion of material in a small cup.

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For further Informations contact us !

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