ML Löteinbettmasse

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•••

Soldering investment, plaster-bound

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ML Lötteinbettmasse

soldering investment, plaster-bound

ML Lötteinbettmasse is a volume-resistant, stable and refractory soldering investment, which can be used for all related joints in the dental field, even at high temperatures. The investment material hardens quickly and can be instantly edited with the flame. Embedded workpieces experienced during the soldering operation without permanent expansion or contraction. The soldered objects are extremely accurately. The workpiece can be easily divest again after soldering. Soldering investment can be used with all dental alloys.

Mixing ratio: 100 g / 30 ml (+ / - 1) Distilled water

Processing: water into the mixing bowl and give cause powder by hand thoroughly with a spatula. Mix by hand or machine about 30-45 sec. Fix soldering object, may, after a waiting period of about 10-15 minutes are soldered immediately

Mixing ratio:	100 g / 30 ml (+/- 1) distilled water
Processing time:	3-5 min. (20 °C)
Ready for soldering:	10-15 minutes
Setting expansion:	< 0,15%
Thermal expansion:	1,2% (at 700 °C)
Max Preheating temperature:	750 °C

Storage stability: **ML - soldering** investment is in dry and frost-free areas at least 8 months stored.

Important notes: **ML - soldering investment material contains quartz is therefore not breathe the dust. Note dust !**

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