mega *CERAPINK*

Plattenwachs

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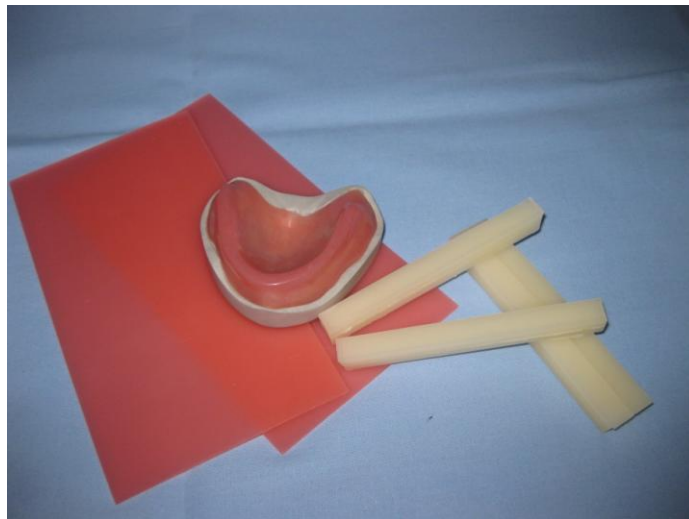
• • •

Modelling wax for denture prosthetics

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mega CERAPINK

Special setup wax

The heat resistant **mega-CERAPINK** setup wax is designed for all prosthetic functions. It's high softening temperature and a melting point around 63 – 65°C makes it ideal for the intraoral wax try-on. It's perfectly suitable for use with warm duplicating gels and for longer phonetic intraoral try-on's.

Composition: Paraffin wax

Characteristics:

- highly stable, not brittle
- melting point 63 – 65°C
- Thickness 1,7 mm, corresponding to a desired thickness of the denture base
- secure fixation of artificial teeth
- does not deform during longer try-on's
- no distortion when using warm duplicating gels in casting systems
- low reset after plastic deformation, after setup of prosthetic works and cooling down
- possible to scrape off without smearing and easy to polish
- odor- and tasteless
- fulfills the high requirements of dental prosthetics

Use: The wax plates can be heated up with a Bunsen burner, in warm water or by using a hot air gun (heat-stick). Thereby the wax plates can be easily adapted to the model. Isolating the plaster model with megasol-NF is sufficient for smooth surfaces or inverse conical parts of the model. An additional moistening of the model is not necessary, which would furthermore cause a contamination of the plaster which could result in an insufficient polymerization of the acrylic. After cooling down, the adapted wax can be easily removed with compressed air. To set up the teeth, the wax needs to be softened at the according position. Additional wax can be applied at any time without discoloration. The final modellation of the wax is carried out after complete cooling of the wax.

Specific details: Reseal package after use to avoid contamination of the material. Store in a cool and dry place. Avoid direct sunlight. Do not use after expiration date. For product stability (3 years) see package label.

Delivery form: 450 g Introduction package Art. Nr. 23-510
2.250 g Laboratory package Art. Nr. 23-520

Class I according to medical device directive 93/42/EEC Annex IX

Storage and safety measures:

Material must not be used after expiration date. Storage temperature must not exceed 25°C [77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station.

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