mega DUR

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For further Informations contact us !

mega DUR

cold curing resin

mega DUR

mega DUR is a Cold curing/polymerizing material based on methyl meth acrylate in the form of powder and liquid. Safe and quick curing by a catalyst system with tertiary amin. High mechanical values and standards, natural shading, very good chemical bonding to dentures. Colour stability due to UV-stabilizer. For relining extensions and repairs of dentures and for orthodontics appliances.

Preparation: Brush alginat isolation onto model plaster. Facet borders and roughen them. The repair borders must be free of wax remainders.

Mixing ratio: 10 units powder and 6 units liquid by volume or 10 units powder and 5 units liquid by weight. Free dosage is possible.

Mixing: Stir powder quickly into the liquid until there is a minor liquid level left. Immediately slowly stir with a slim clean instrument for 15-20 seconds in order to avoid air inclusions.

Working: Place the acrylic material, which capable of dripping, onto the roughened denture area, evenly and with excess. After the flowing phase, in the plastic condition, the mega-DUR material may not be modelled any more. In order to avoid pores and bubbles, thick material areas must be applied in layers. Before applying the new dough, the acrylic must be cured and cold. A safe curing is only secured under pressure at a water temperature of appr. 30-40 °C.

Polymerization: **mega DUR** cures at room temperature of 23 °C in appr. 10-12 min. Polymerization in the pressure polymerizing unit or water pressure pot (water temperature 20 °C) excludes to a large extent the development of pores and bubbles which occur due to exothermical heat. The repair areas can be cut and polished immediately after curing.

Colours: pink, pink-veined, clear

Packing: 0,5 kg / 1 kg / 5 kg / 20 kg # 0,5 Lit. / 1 Lit. / 5 Lit.

Storage and safety measures:

Material must not be used after expiration date. Storage temperature must not exceed 25°C [77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station.

megadental is certified in accordance to ISO 9001 and guarantees efficient quality through maintaining a deliberate quality management system. Product properties as mentioned in this instruction can only be guaranteed when the procedural steps are carried out properly. The user is responsible for correct handling of the product. Any defective results cannot be accounted for, because megadental has no influence on the subsequent processing by the user. Any claims for damages can solely be granted for the value of our goods.

according to medical device directive 93/42 EEC annex IX **class IIa** for removable dentures

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megadental GmbH • Seeweg 20 • D-63654 Büdingen • Tel.: +49(0)6042-9755.0 • FAX +49(0)6042-9755.20