mega *GEN LC*

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For further Informations contact us !

mega GEN LC

mega GEN - LC burns out without residue. It is a light curing, gel-like / pasty sculpting resin for dental applications. It can be either applied directly from the syringe or taken from the container for processing by using a spatula. The material burns out without residue, can be cured using any standard light curing device (320-500 nm) and can be combined with any standard wax. Directly after the application **mega GEN - LC** does not run, the modellations are stable, precise and may be removed without distortion.

- Contouring precision attachments, telescope and conical crowns etc.
- Contouring of individual implant abutments (extraoral)
- Connecting of cast units (extraoral)
- Contouring adhesive bridges, inlays and onlays
- Contouring of crowns and bridges for pantographic transmission

Instructions for use: After removing the seal cap, attach the application tip supplied for precise application. These are intended for single use. Then seal the slide syringe again. The paste can be sculpted and contoured easily by hand due to its plastic consistency. Build up the pattern in layers no thicker than 1 mm and perform interim curing of these layers for 10 to 60 seconds (depending on the intensity of the curing device) with a light spectrum of 320-500 nm.

Polymerisation The average curing times are 90 sec with strobe lamps. 2 min in devices using halogen lamps and 3,5 – 5 min in divices using UVA-bulbs.

Important: Do not remove the inhibition layer from the surface; it provides an optimum bond between the layers and prevents stresses in the modelling resin. The inhibition layer should only be removed from the surface using alcohol or acetone after contouring of the pattern is complete or if wax is to be added to the pattern.
To obtain optimum casting results, complete removing of the inhibition layer is essential. On large-volume patterns, object should be complemented with a thin wax coating.

Polymerising layers thicker than 1 mm can result in incomplete curing, excessive thermal release and stresses and does not guarantee an exact fit of the casting. **mega GEN - LC** should be applied and polymerised in layers of maximum 1 mm thickness to keep shrinkage as low as possible during curing.

Please note that curing times depend on the emission power of the lamps used in light curing devices. Correct maintenance of these devices as well as regular replacement of illuminants account for a constant curing. Please also note the instructions of the device manufacturer.

Density: ~1,1 g / cm³

Package Set with 3 syringes containing 3g and 9 Cannulas or 30 g container

Storage and safety measures:

Material must not be used after expiration date. Storage between 4 - 25°C [12 - 77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition.

Sensitiv people can be an awareness of the product. In this case we recommend to cease the use. When preparing use a suction extraction system.

megadental is certified in accordance to ISO 9001 and guarantees efficient quality through maintaining a deliberate quality management system. Product properties as mentioned in this instruction can only be guaranteed when the procedural steps are carried out properly. The user is responsible for correct handling of the product. Any defective results cannot be accounted for, because megadental has no influence on the subsequent processing by the user. Any claims for damages can solely be granted for the value of our goods.

according to medical device directive 93/42 EEC annex IX **class IIa** for fixed and removable dentures.

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