mega PRESS LV

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mega PRESS LV

Cold curing resin for partials
with long processingtime

mega PRESS LV

Material: **mega PRESS LV** cold curing resin on a Methylmethacrylatbasis, with long processing time. For the completion of partials, for Underlining, Repairs and for the shaping of the edges of prosthesis. **mega PRESS LV** is color-stable due to the catalysersystem. The advantages are in the high mechanical values, the secure and fast hardening. Without Amin and Cadmium.

Working: Plaster model should be watered copiously. This is important, for do not have Bubbles during the polymerisation in the resin. The watered plaster model must be coated with Alginat insulation (for exmp. **megasol**).

Mixingratio: **10 g Powder : 5-7 g Liquid** (or as desired) Liquid into mixingbowl then add the powder

Working time: stir in a fast motion, until it is a creamy dripping consistency. Pour the mixed resin into the pre-models. The pourable 1-3 Min., plastic phase 4-10 Min. By Room 20-22°C !

Polymerisation: Latest after 8-12 minutes must begin the Polymerisation by 45-55°C / 2-3 bar / 15-20 Minutes / Attention! - by **mega-Cast system** with dublicating-gel (**megafeel Extra**) polymerisation 30 minutes

Colours: pink, pink-veined, light-pink, light-pink-veined, pink-opak, V5, clear

Packing: 0,5 kg / 1 kg / 5 kg / 20 kg 0,5 Lit. / 1 Lit. / 5 Lit.

	Lot number		manufacturer
	Expiry date		Xi, irritant
	Read instructions for use		F, Highly inflammable
	Recommended storage temperature		Xn, harmful to health

Storage and safety measures:

Material must not be used after expiration date. Storage temperature must not exceed 25°C [77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station.

megadental is certified in accordance to ISO 9001 and guarantees efficient quality through maintaining a deliberate quality management system. Product properties as mentioned in this instruction can only be guaranteed when the procedural steps are carried out properly. The user is responsible for correct handling of the product. Any defective results cannot be accounted for, because **megadental** has no influence on the subsequent processing by the user. Any claims for damages can solely be granted for the value of our goods.

according to medical device directive 93/42 EEC annex IX **class IIa** for fixed and removable dentures
according to DIN EN ISO 20795 Type 2 Group 1

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