mega *RESIN 2000*

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For further Informations contact us !

mega *RESIN* 2000

Super Transparent Display Acrylic for demonstration models

mega-RESIN 2000 is a castable display acrylic based on PMMA/PEMA, super transparent for crystal clear demonstration models and therefore perfectly suitable for the presentation of implants in dentistry. The material is based on a catalyst system without tertiary amines and thus maintains excellent color stability and transparency.

Preparation:

The duplication mould should be clean and free of grease, the material can be processed in either silicone or gel (agar-agar).

Directions for use:

Mixing: Place required amount of liquid into the mixing cup, then add the according amount of powder while stirring constantly. The chosen mixing ratio should allow enclosed air bubbles to discharge from the material. We recommend a mixing ratio between **10:5** and **10:6** (Powder:Liquid). Dosing individually is possible.

Processing: The casting phase is 1 minute at maximum, it is recommended to process the material quickly. Storing the two components in a cool space prior to processing extends the processing time.

Pour the material into the mould during its casting phase. Locked-in air bubbles can be removed by using a dentistry instrument before placing the mould into the pressure polymerization unit immediately.

Important: The polymerization is carried out inside a pressure polymerization unit for 20 minutes at a pressure of 2-3 bar in cold water.

Important information: Containers should be kept closed after use in order to prevent contamination. Keep in a dry space at room temperature, avoid direct sunlight. The material should not be used after expiration date (3 years) indicated on the package.

Package sizes: 500 g / 1.000 g Powder; 500 ml / 1000 ml Liquid

Storage and safety measures:

Material must not be used after expiration date. Storage temperature must not exceed 25°C [77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station.

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