mega *Provi Implant*

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For further Informations contact us !

megadental GmbH • Seeweg 20 • D-63654 Büdingen • Tel.: +49(0)6042-9755.0 • FAX +49(0)06042-9755.20
e-Mail: info@megadental.de • Internet: <http://www.megadental.de> Stand: 08.2012

mega Provi Implant

for fabricating of temporary crowns with micro pearls

mega Provi Implant

Powder and liquid for fabricating temporary crowns and bridges. **mega Provi Implant** is based on polymethylacrylate and does not contain cadmium. As **mega Provi Implant** does not contain tertiary amines, it does not reach high temperatures during curing. Therefore, it only exhibits minimal shrinkage and does not distort. **mega Provi Implant** is an auto-curing resin which only contains 0,8 % of residual monomer and is recommended for long term provisionals in implantology.

Fabrication of a laboratory-made crown or bridge

1. To achieve an ideal provisional, the laboratory should be provided with three impressions. It should be an impression before and after the preparation, as well as one counter-bite.
2. After fabrication of the working models, preferably made of a class III dental plaster, attach the stump model and the counter bite in the articulator.
3. If only one stump model is available, close the gaps with a wax-up of acrylic teeth. In this case mounting is not applicable.
4. Isolate the situation model with a thin layer of silicone spray or Vaseline, or use the stump model with a wax-up.
5. We recommend covering the isolated model by using a 0,5mm thermoforming foil, and cutting off the overlapping edges.
6. Clean the situation model with steam; afterwards isolate it with alginate based isolation. When using the stump model, remove the wax-up. After cleaning, gaps and fissures close to the stump need to be blocked out and isolated as well.
7. Mix the powder and liquid component with a mixing ratio of 10 : 4–6 g (Powder : Liquid) to a still flowable consistency and pour the material into the thermoforming foil where the stumps will be situated. Reposition the thermoforming foil exactly on the isolated stump model and remove excessive material.
8. The polymerization can be carried out with or without a pressure polymerization unit. When using a pressure polymerization unit, cure mega Provi Implant for 12-15 min. in warm water (37-45°C) and a pressure of 2-4 bar. Otherwise the material cures at room temperature within approx. 20 min.
9. Afterwards remove the provisional from the model and finish it. Make sure to create sufficient space for the cement! Verification of the occlusion can be done using the counter bite. The cleaned and polished provisional can be incorporated by using regular provisional cement.

Mixingratio: 10 parts powder and 4-6 parts liquid.

colours: S00, SW, SE1, SE2, SE3, Light-Blue, DA1, DA2, DA3, DA3,5, DA4 DB1, DB2, DB3, DC2 and DD3.

Packing: 100g / 500 g # 0,1 Lit. / 0,5 Lit.

Storage and safety measures:

Material must not be used after expiration date. Storage temperature must not exceed 25°C [77°F]. Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station.

according to medical device directive 93/42 EEC annex IX **class IIa** for fixed and removable dentures

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	Lot number		manufaturer
	Expiry date		Xi, irritant
	Read instructions for use		F, Highly inflammable
	Recommended storage temperature		Xn, harmful to heath

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Megadental GmbH • Seeweg 20 • D-63654 Büdingen • Tel.: +49(0)6042-9755.0 • FAX +49(0)6042-9755.20